

# EtchMaster Adaptor Cleaning Instructions

## **Disinfection instructions:**

Groman Dental recommends using a disinfectant on the adapter body between patients or using a single use disposable barrier.

To disinfect the EtchMaster adapter, generously spray disinfectant on a clean towel and wipe all surfaces. Discard used towel. Dry with a clean cloth.

Never spray disinfectant solution directly on the adapter.

When using a plastic sleeve over the EtchMaster adapter, pierce the sleeve to the bulb portion of the powder tip to limit direct patient contact to the adapter.

Do not immerse handpieces in ultrasonic bath.

## **DO NOT AUTOCLAVE**

For further help or assistance, contact Melanie on: 01359 220133, or email: [info@dpsdental.co.uk](mailto:info@dpsdental.co.uk)